

Verdünnungstabelle - Marken

Manufacturer/ Brand	Vial [units]	Solvent Added Sodium Chloride [ml]	Reconstitution Solution [units/0.1 ml]	3Dose Syringe - green - orange	3Dose Plunger Setting - black - blue	Number of units per click 1 Click =
Ipsen/Galderma Azzalure® Dysport®	125 Speywood units	0.625 ml	20 Speywood units / 0.1 ml	Green 3DG125	black: 0.025	5 Speywood units
	300 Speywood units	1.5 ml			blue: 0.05	10 Speywood units
	500 Speywood units	2.5 ml				
	125 Speywood units	1.25 ml	10 Speywood units / 0.1 ml	Green 3DG125	blue: 0.05	5 Speywood units
Allergan Botox® Vistabel®	50 units	1.25 ml	4 units / 0.1 ml	Green 3DG125	black: 0.025	1 unit
	100 units	2.50 ml			blue: 0.05	2 units
Ipsen/Galderma Alluzience®	125 Speywood units in 0.625 ml		20 Speywood units / 0.1 ml	Green 3DG125	black: 0.025	5 Speywood units
					blue: 0.05	10 Speywood units
Xeomin® Bocouture®	50 units	1.0 ml	5 units / 0.1 ml	Orange 3DO100	black: 0.02	1 unit
	100 units	2.0 ml			blue: 0.04	2 units
	50 units	1.25 ml	4 units / 0.1 ml	Green 3DG125	black: 0.025	1 unit
	100 units	2.50 ml			blue: 0.05	2 units